


# Wheelchair Tray Installation Instructions

1. Remove the plastic protective cover from both sides of the tray
2. Place tray on a flat surface with the top facing up (if there is no rim on the tray it will not matter which side is up)
3. Remove the adhesive cover from the adhesive tray straps
4. Place the Velcro straps approximately 2-3 inches from the bottom of the tray with the adhesive side facing the tray. Align the outside edge of the tray with the seam (where Velcro and adhesive meet) on the Velcro straps. Press down on the adhesive to make a strong bond to the tray
5. Trim the excess adhesive from the strap
6. Place the tray on the wheelchair and wrap the Velcro straps tight around the armrests
7. To remove tray, simply lift up on the Velcro straps

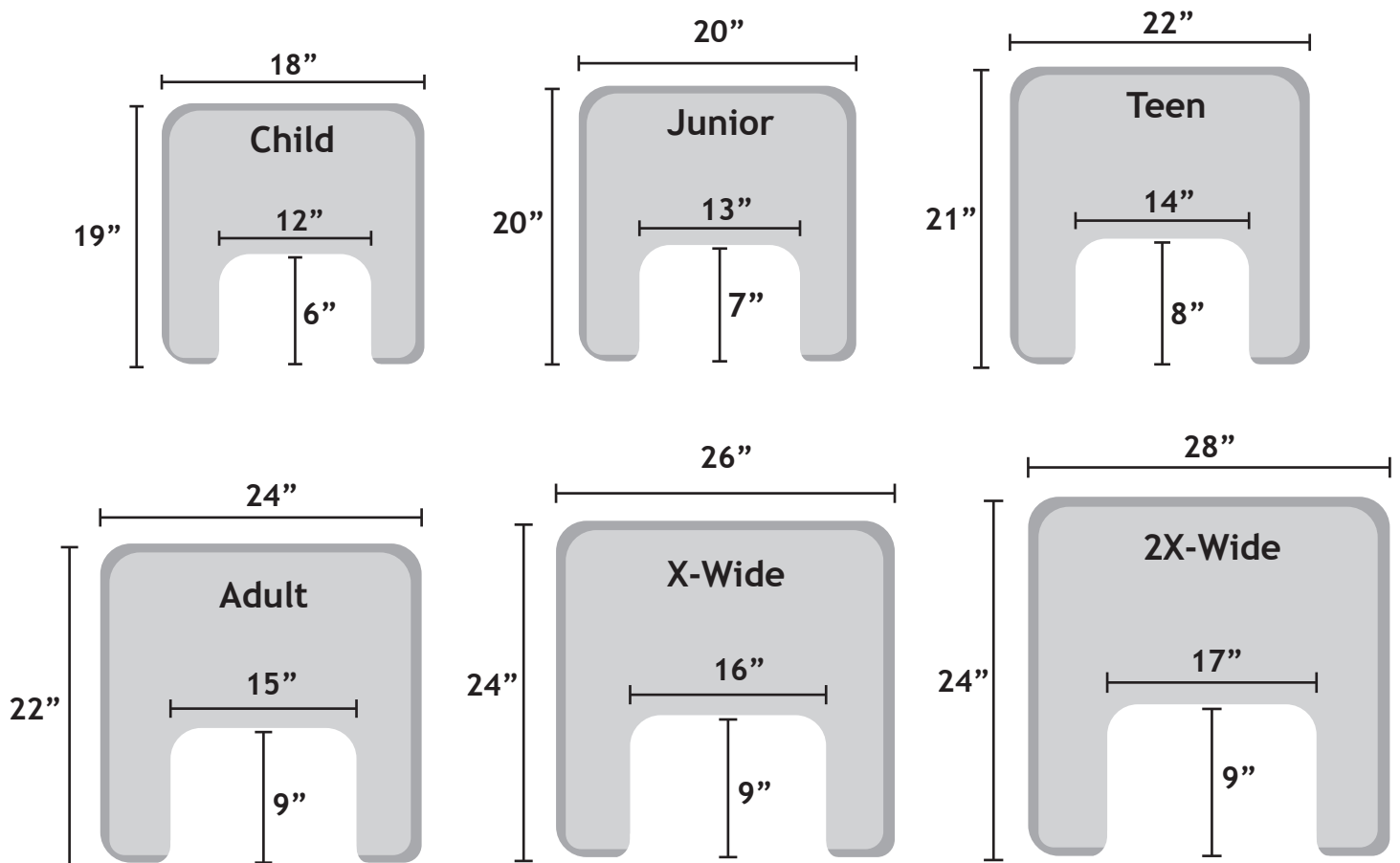

## Tips

Before beginning with step 3, trial fit the tray on the wheelchair with your loved one sitting in the chair. This will insure that there will be no interference with the strap placement and the wheelchair